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Diagnostický disk VAN 5 µg obsahuje 5 µg vankomycínu a je určený k testovaniu rezistencie mikroorganizmov k vankomycínu.

PRINCÍP

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Dátum poslednej revízie: 20.05.2022

SUMMARY AND EXPLANATION

Diagnostic disc VAN with content of 5 µg vancomycin is intended for detection strains with resistance to vancomycin.

PRINCIPLE

Antibiotics diffuse from disc to agar, what creates partly selective zone inhibiting growth of native microbiome.

DIAGNOSTIC KIT CONTAINS

- tube with diagnostic discs - min. 500 pcs (Ref. 6036-500)
- min. 50 pcs (Ref. 6036-50)

WARNINGS AND SECURITY PRECAUTIONS

- **For *in vitro* diagnostics use.**
- **For professional use only**
- Follow the instructions exactly!
- All samples and inoculated products have to be considered as potentially infectious and obvious safety valid regulations must be respected while handling.
- Do not use product out of expiration.
- Before use, check if the packaging is intact. Do not use damaged kit.
- Leftovers of expired or devaluated product liquidate according to regulations of your country. Packaging materials are recyclable.

STORAGE

Keep discs in dark at temperatures between +2 and +8 °C.

REQUIRED MATERIALS

- Petri dish with blood agar
- loops, burner, tubes and other basic equipment of microbiological laboratory

RECOMMENDED PROCEDURE

1. Spread tested strain on Petri dish.
2. Place sterile disc VAN on inoculated Petri dish, press it gently, so that whole surface of disc is in contact with medium. Incubate for 18-24 h in CO₂ atmosphere at temperature 35 ± 2 °C.

EVALUATION AND INTERPRETATION

Microorganisms sensitive to vancomycin do not grow in inhibition zone. Consider actual information stated at www.eucast.org, or www.clsi.org

For enterococci, the strain can be considered sensitive if the area is less than 12 mm.

Functionality of diagnostic discs can be verified by these control strains:

Bacterial strain	CCM / ATCC	Inhibition zone
<i>Enterococcus faecalis</i>	4224 / 29212	10-16 mm

CCM: Czech Collection of Microorganisms, Masaryk University Brno, Kamenice 5, 625 00 Brno, Czech Republic, tel. +420549491430, e-mail: ccm@sci.muni.cz

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