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Dátum poslednej revízie: 1. 2. 2018

SUMMARY AND EXPLANATION

Assimilation diagnostic discs are designed for identification of clinically significant species of yeasts.

PRINCIPLE

Principle is based on ability of yeasts to assimilate carbohydrates and other chemicals diffused from disc into agar, what causes that grow of strain is stimulated around assimilation disc.

DIAGNOSTIC KIT CONTAINS

- tube with diagnostic discs - 500 pcs / 50 pcs

Ref.	Name	Description	Ref.	Name	Description
6001-50	ARA	Utilization of arabinose	6011-500	MLZ	Utilization of melzitose
6001-500			6012-50	RAF	Utilization of raffinose
6002-50	CEL	Utilization of arabinose	6013-50	RHA	Utilization of rhamnose
6002-500			6014-50	RIB	Utilization of ribose
6003-50	ERY	Utilization of erythritol	6015-50	SOR	Utilization of sorbitol
6003-500			6016-50	SUC	Utilization of sucrose
6004-50	GAL	Utilization of galactose	6017-50	TRE	Utilization of trehalose
6004-500			6018-50	XYL	Utilization of xylose
6005-50	GLU	Utilization of glucose	6019-50	NIT	Utilization of KNO ₃
6005-500			6020-50	PEP	Utilization of peptone
6006-50	INO	Utilization of inositol	6021-50	URE	Utilization of urea
6006-500			6022-500		
6007-50	LAC	Utilization of lactose			
6007-500					
6008-50	MNS	Utilization of mannose			
6008-500					
6009-50	MLB	Utilization of melibiose			
6009-500					
6010-50	MLT	Utilization of maltose			
6010-500					
6011-500	MLZ	Utilization of melzitose			

WARNINGS AND SAFETY PRECAUTIONS

- For *in vitro* diagnostics use.
- For professional use only.
- Follow the instructions exactly!
- All samples and inoculated products have to be considered as potentially infectious and obvious safety valid regulations must be respected while handling.
- Do not use product out of expiration
- Before use, check if the packaging is intact. Do not use damaged kit.
- Leftovers of expired or devaluated product liquidate according to valid regulations. Packaging materials are recyclable.

STORAGE

Keep diagnostic discs in dark at temperatures between +2 and +25 °C.

REQUIRED MATERIALS

- C-agar for auxanogram for discs with ref. 6001-6018, N-agar for discs with ref. 6019-6021
- sterile Petri dish, sterile inoculation loop, water bath
- saline 1 ml

RECOMMENDED PROCEDURE

1. Take one full inoculation loop of yeast mass from 24 hours old culture and suspend it in tube with sterile saline (1 ml). Suspensions must be prepared immediately before its usage.
2. Mix prepared suspension with agar for auxanogram, which was melted before, in water bath (t < 52 °C). Pour agar on sterile Petri dish, than, after solidification, put discs on agar (max. 7 on Petri dish). Press discs gently, so that whole surface of discs is in contact with medium. Incubate it in thermostat for 24 h at temperatures 25-30 °C.

EVALUATION AND INTERPRETATION

As positive result is interpreted grow of colonies around diagnostic discs seen as round zone. In case of negative result grow of colonies around disc is not visible.

Functionality of discs can be verified on control strains (C.S.):

1. *Candida guilliermondii* CCM 8321 / ATCC 90877
2. *Cryptococcus neoformans* CCM 8312 / ATCC 90112
3. *Klebsiella pneumoniae* CCM 5852 / ATCC 13882

C. S.	A	C	E	G	I	L	M	M	M	R	R	R	S	S	T	X	N	P	U
	R	A	R	A	L	N	A	L	L	L	L	L	L	L	L	L	L	L	L
	A	A	L	L	L	O	C	S	B	T	Z	F	A	B	R	C	U	R	E
1.	+	+	-	+	+	-	+	+	+	+	+	+	+	+	+	+	+	+	+
2.	-	-	-	+	+	+	-	+	-	+	+	+	+	+	+	+	+	+	+
3.	+	+	-	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+

CCM: Czech Collection of Microorganisms, Masaryk University Brno, Kamenice 5, 625 00 Brno, Czech Republic, tel. +420549491430, e-mail: ccm@sci.muni.cz

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