Alternatívna matrica pre MS MALDI-TOF / Alternative matrix for MS MALDI-TOF / Alternativní matrice pro MS MALDI-TOF / MS MALDI-TOF –hoz való alternatív mátrix

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Dátum poslednej revízie: 21.9.2017

SUMMARY AND EXPLANATION

Optimized matrix substance is intended for preparation of matrix solution for mass spectrometry by simple addition of organic solvent. Amount in microtube corresponds to 2,6 - 2,8 mg of substance.

DIAGNOSTIC KIT CONTAINS

- 10 microtubes with substance (alfa-cyano-4-hydroxy cinnamic acid)
- Organic solvent (50 % acetonitrile, 45 % water, 5 % trifluoroacetic acid)
- information leaflet

WARNINGS AND SECURITY PRECAUTIONS

- **For in vitro diagnostics use.**
- **For professional use only.**
- Follow the instructions exactly!
- All samples and inoculated products have to be considered as potentially infectious and obvious safety valid regulations must be respected while handling.
- Before use, check if the packaging is intact. Do not use damaged kit. Leftovers of expired or devaluated reagent liquidate according to valid regulations. Packaging materials are recyclable.

STORAGE

Diagnostic kits are delivered in multilayer packets on base of aluminium and organic polymers. Part of each packet is silica gel desiccant. Store kits at temperature from +2 to +25°C. Expiration date is placed on each packaging. Store opened product at dark and dry place, with temperature under +25°C. Product can be stored one week in such conditions (or to date of expiration, in case when it passes earlier).

RECCOMENDED PROCEDURE**Preparation of inoculum****DIRECT INOCULATION METHOD**

- Dissolve content of microtube in 280 µl of organic solvent and homogenize. **Total dissolution of crystals isn't needed.** The presence of crystals indicate creation of saturated solution, which is necessary for properly function of matrix. Creation of crystals in original Bruker's matrix is caused by evaporation of organic solvent, which leads to incorrect function of product. These two cases of insolubility of crystals are different!
- Spread biological material in thin homogeneous layer on MALDI plate
- Cover sample with 1 µl of matrix.
- Let dry at laboratory temperature
- Analyse in device for mass spectrometry MALDI

Tip: Upper mentioned method, can be modified: spread one colony at one position on MALDI plate and cover it with 1 µl of 70 % solution of formic acid after drying. When dried, cover sample with 1 µl of prepared substance - matrix for mass spectrometry MALDI and after drying analyse in device for mass spectrometry MALDI.

EXTRACTION METHOD

- Spread 1 µl of extracted material in homogenous layer at one position of MALDI plate
- Dry at laboratory temperature
- Cover sample with 1 µl of matrix substance
- Dry at laboratory temperature
- Analyse in device for mass spectrometry MALDI

WASTE LIQUIDATION

Work with material as with potentially infectious agents. Liquidate leavings according to internal procedures and legislative of your country.

Kit components do not contain dangerous chemicals.

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