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Dátum poslednej revízie: 09.11.2023

SUMMARY AND EXPLANATION

IND reagent is used for generation of colour in test IND in diagnostic kits. Detailed description of work with reagent IND and interpretation of result of test is part of particular kit manual.

PRINCIPLE

By the action of the enzyme tryptophanase, indole is created and detected with IND reagent.

DIAGNOSTIC KIT CONTAINS

- IND reagent min. 18 ml (Ref. 3002)
- information leaflet

WARNINGS AND SAFETY PRECAUTIONS

- **For *in vitro* diagnostics use.**
- **For professional use only.**
- Follow the instructions exactly!
- All samples and inoculated products have to be considered as potentially infectious and obvious safety valid regulations must be respected while handling.
- Before use, check if the packaging is intact. Do not use damaged kit.
- Leftovers of expired or devaluated reagent liquidate according to regulations of your country. Packaging materials are recyclable.

Kit contains 50% of ethanol, considering this, it is classified as highly flammable and health hazardous.

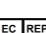
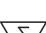
H226 Flammable liquid and vapour
H302 Harmful if swallowed
H315 Causes skin irritation
H318 Causes serious eye damage

P210 Keep away from heat/sparks/open flames/hot surfaces - No smoking.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P270 Do not eat, drink or smoke when using this product.
P305+P351+P338
IF IN EYES: Rinse cautiously with water for several minutes.
Remove contact lenses, if present and easy to do. Continue rinsing.
P302+P352 IF ON SKIN: Wash with plenty of soap and water.
P301+312 IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell.
P304+P340 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.
P332+P313 If skin irritation occurs: Get medical advice /attention.
P370+P378 In case of fire: Use water or CO2 for extinguishing.
P403+P233+P235 Store in a well - ventilated place. Keep container tightly closed. Keep cool.

STORAGE

IND reagent can be stored in temperatures between +2 and +8°C. Expiration date is showed on label of each package.

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